


ThinPrep® Pap Test™ Quick Reference Guide

Endocervical Brush/Spatula Protocol

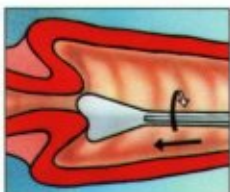


Obtain...

...an adequate sampling from the ectocervix using a *plastic* spatula.



Rinse...

...the spatula into the PreservCyt® Solution vial by swirling the spatula vigorously in the vial 10 times. Discard the spatula.

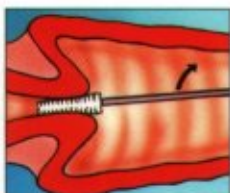

Obtain...

...an adequate sampling from the endocervix using an endocervical brush device. Insert the brush into the cervix until only the bottommost fibers are exposed. Slowly rotate $\frac{1}{4}$ or $\frac{1}{2}$ turn in one direction. **DO NOT OVER-ROTATE.**



Rinse...

...the brush in the PreservCyt Solution by rotating the device in the solution 10 times while pushing against the PreservCyt vial wall. Swirl the brush vigorously to further release material. Discard the brush.



Tighten...

...the cap so that the torque line on the cap passes the torque line on the vial.



Record...

...the patient's name and ID number on the vial.
...the patient information and medical history on the cytology requisition form.



Place...

...the vial and requisition in a specimen bag for transport to the laboratory.

The ThinPrep® Pap Test™
Clear and simple.